CE
0051

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HYPLUS is a medical device with activity to prevent and treat skin lesions caused by pressure ulcers. The combination of its active ingredients also protects the skin by keeping it lubricated and hydrated.

Its formulation promotes skin renewal and accelerates wound or bedsores healing, enhances skin's barrier function preventing their occurrence, and keeps the skin healthy thanks to its antioxidant properties.

Composition

Hypericum extract (*Hypericum perforatum* L.) 2%, olive oil (*Olea europaea* L.) 98%.

Intended use and indications

HYPLUS is indicated for prevention and treatment of stage I (skin with redness) and stage II (shallow open ulcer or blister) pressure ulcers in adults (18 years and older). It is also indicated for helping in the treatment of other skin lesions such as atopic dermatitis or eczema in adults (18 years and older).

HYPLUS is a medical device for topical use which stimulates epidermal regeneration thus accelerating wound or ulcer healing process, improves the appearance of post-scar tissue and promotes skin's barrier function. Thanks to its lipophilic characteristics, it achieves an isolating-protective action on the area of application.

Instructions for use

Apply directly on clean and dry skin, in enough quantity to cover the area to be treated. Repeat the application 2 times a day until visible improvement of the lesion, for no longer than 30 days. HYPLUS must be applied with proper hygiene, with the use of gloves and spreading the product using the fingers. Pressure ulcers treatment must be accompanied by regular repositioning of the patient.

Adverse effects

Local effects (temporary irritation) may occur in the area of application. If there is any discomfort after applying this product to the skin, stop using it and consult your physician or pharmacist. In case of adverse effects related to the product, report them to the manufacturer and/or distributor. In case of serious incidents, report them to the competent health authorities.

Contraindications

Do not use in children or adolescents (0-18 years). Under normal conditions of use, no contraindications have been reported. In case of hypersensitivity to any of the components, stop the treatment and contact your physician.

Warnings and precautions

For external skin use only (topical). Only apply this product in the area of skin affected and for its intended use and population. Do not apply in case there is an additional lesion or condition different to those included in the indication in the affected area. Do not apply together with other topical products or medicines. Not recommended for use during pregnancy and breastfeeding. Keep out of the sight and reach of children. Do not use if the package is damaged. Use gloves for its application. Wash your hands before and after each use. Aim the applicator to the area to be treated, avoiding contact with other areas. Do not aim the applicator to the eyes, in case of contact with the eyes, rinse with plenty of water. Do not use in mucous areas (eyes, mouth, genitals). Do not swallow. Intended to be used by healthcare professionals or patients at home.

Storage conditions

Do not use after the expiry date printed on the package. Do not use for more than 6 months after the first use of the product. Do not store above 25 °C. Store in the original package, protected from heat and direct sunlight. No special requirements for the disposal of this device. Empty bottles are disposed according to local regulations. Ask your pharmacist.

Formats

This medical device is available in the following formats: 30 ml and 100 ml.

Date of issue: October 2025. Revision 06

Especialidades Farmacéuticas Centrum, S.A.
C/ Sagitario, 14 - 03006 - Alicante
MADE IN SPAIN
ASACPHARMA / www.asac.net

120809-03